Contents

Wall Mounting User Instructions.....	1
Инструкции за потребителя за конзолата за стениен монтаж.....	5
壁掛支架使用说明.....	10
壁掛支架使用說明.....	14
Korisničke upute za montažu na zid.....	18
Užívateľské pokyny pro montáž na stěnu.....	22
Brugervejledning til vægmontering.....	27
Gebruikersinstructies voor wandbevestiging.....	31
Seinakinnituse kasutusjuhend.....	36
Seinätelineen käyttöohjeet.....	40
Instructions d'utilisation du support de montage mural.....	44
Gebrauchsanweisung für Wandhalter.....	49
Οδηγίες χρήσης επιτοίχιας βάσης.....	54
Fali tartóelem használati útmutató.....	59
Petunjuk Penggunaan Dudukan Dinding.....	64
Istruzioni per l'uso del supporto a parete.....	69
Қабырғаға арналған бекітпе бойынша пайдаланушыға арналған нұсқаулар.....	74
벽면 장착 사용자 지침.....	79
Sienas stiprinājuma lietošanas instrukcija.....	83
Istruzioni per l'uso del supporto a parete.....	87
Bruksanvisning for veggfeste.....	92
Instrukcja użytkowania mocowania ściennego.....	96
Instruções do usuário para montagem na parede.....	101
Instruções de utilização do suporte de parede.....	106
Instrucțiuni de utilizare a suportului vertical.....	111
Инструкции по эксплуатации настенного крепежа.....	115
Korisnička uputstva za držač za zid.....	120
Návod na montáž na stenu.....	125
Instrucciones para el usuario para el soporte de montaje en pared.....	129
Instruktioner för väggmontering.....	134
Duvara Montaj Kullanıcı Talimatları.....	138
Інструкції користувача підвісу.....	142
Hướng dẫn sử dụng Giá gắn tường.....	147

Wall Mounting User Instructions

Dawnwatch Product Number: FGE#5937979

All specifications are subject to change without notice.

Suzhou Dawnwatch Medical Equipment Co., Ltd.

Tel: +86 512 66032558 66171878

Fax: +86 512 66301668

Date: 2023/04/23


Warnings

- For continued safe use of this equipment, it is necessary to follow these instructions. However, the instructions in no way supersede established medical practices concerning patient care.
- Do not exceed the maximum load weight of each part.
- To avoid personal injury to users or any other persons moving in the vicinity of the cables, route all the cables in such a way that they do not present a tripping hazard.
- Do not install the wall mounting above where the patient's head will be.
- After installing the wall mounting, always check that it is securely attached. Otherwise, there is a risk that it could fall and injure the patient or become damaged.
- No modification of this equipment is allowed.

Introduction


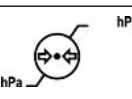
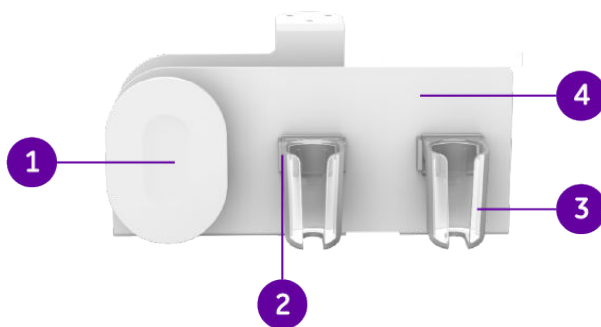The wall mounting provides a means of mounting monitor-compatible supplies or accessories on the arm.


Item number	Description
1	Cable management

Item number	Description
2	Wall mounting buckle (two pieces)
3	75mm holder (limited weight < 1kg) (two pieces)
4	Wall mounting base (limited weight < 3kg)

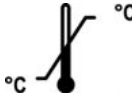Equipment Symbols List

	Temperature limitations.
	Humidity limitations.
	Atmospheric pressure limitations.
	European Union Declaration of Conformity.
	The maximum load of the wall mounting base is 3kg.

Installation

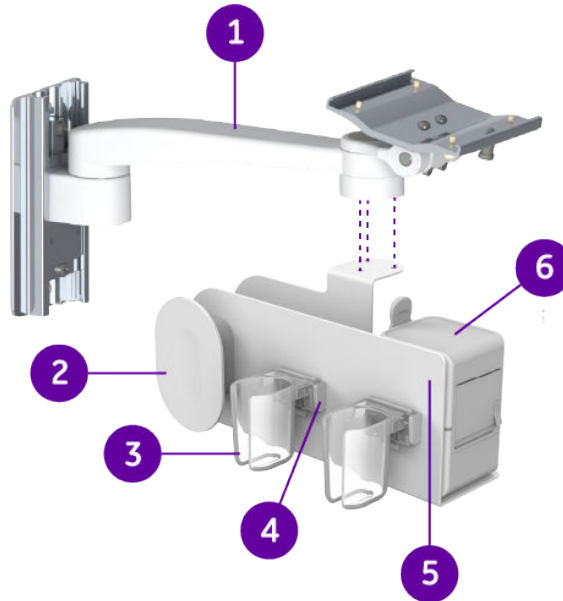Item number	Description
1	Monitor mounting arm: For more information about the arm, refer to the instructions for the arm.
2	Cable management: Organize and secure cables such as a NIBP cuff hose.

Item number	Description
3	75mm holder: Secure accessories such as a thermometer.
4	Wall mounting buckle: Fixes the 75mm holders to the wall mounting.
5	Wall mounting base: A metal fixing part used to fix other parts.
6	Recorder (optional): Must be ordered separately.

1. Align the three holes in the wall mounting to the corresponding three screw holes in the arm.
2. Use a screwdriver to secure the wall mounting to the arm with three #10-32×2/4 socket head cap screws (torque 20kgf.cm +/-10%).
3. For each of the two 75mm holders, pinch the hook on the holder and hang it on the wall mounting buckle. Push down each holder until it clicks.
4. To assemble the recorder, align the recorder to the rail and push down the recorder until it clicks.
5. Ensure that the screws are tightened and the wall mounting has been secured to the arm. Gently shake the arm by hand and make sure that the wall mounting does not shake.

Maintenance

Please perform regular inspections of the equipment according to facility requirements or local regulations. Always ensure that the screws are tightened.

Disposal Instructions

The product described in this manual, as well as its accessories and packaging, must be disposed of according to local environmental and waste disposal regulations. Always consider your hospital guidelines as well.

Cleaning and Disinfection

The following table lists the permitted cleaning and disinfection agents for the equipment.

Item number	Product	Manufacturer	Type of disinfectant	Active ingredients
1	70-96% ethanol	Generic	Alcohol	Ethanol (CAS 64-17-5)
2	Super Sani-Cloth® Germicidal Disposable Wipe	PDI	Quaternary ammonium and alcohol	Isopropyl alcohol (CAS 67-63-0) 55.5% Quaternary ammonium compounds, C12-18-alkyl [(ethylphenyl) methyl] dimethyl, chlorides (CAS 68956-79-6) 0.25% n-alkyl dimethyl benzyl ammonium chloride (CAS 68391-01-5) 0.25%
3	Bleach max. 5.25% by volume mixed with water in the ratio of 1:10	Generic	Chlorine	Sodium hypochlorite (CAS 7681-52-9) 0.525%
4	Sani-Cloth Bleach Germicidal Disposable Wipes	PDI	Chlorine	Sodium hypochlorite (CAS 7681-52-9) 0.63%

Item number	Product	Manufacturer	Type of disinfectant	Active ingredients
5	Oxivir TB wipes	Diversey	Hydrogen peroxide	Benzyl alcohol (CAS 100-51-6) 1-5% Hydrogen peroxide (CAS 7722-84-1) >0.1 - <1%
6	Getinge Clean*	Getinge	Enzymatic or neutral pH detergents	Enzymatic or neutral pH detergents

*This agent can only be used for cleaning.

Ensure the wall mounting is clean before starting to use. Follow these instructions to clean and disinfect the wall mounting. Always consider your professional healthcare facility guidelines as well.

1. Remove all accessories.
2. Put on new disposable gloves.
3. Get a new wipe or dampen a soft, low-linting cloth with one of the permitted detergents.
4. Wipe the equipment's exterior surface. Remove any dirt by wiping the equipment until all dirt and organic matter has been visibly removed.
5. If required, take off the holders for cleaning.
6. Inspect the equipment to ensure the complete removal of dirt from surfaces and movable parts. If visible dirt remains, repeat cleaning procedure until the equipment is thoroughly clean.
7. If required, manually rinse the equipment by wiping it for 30 seconds using a sterile, soft, low-linting cloth saturated with lukewarm water (temperature range 27 to 44°C, 81 to 111°F).
8. Allow the equipment to air dry until it is visibly dry. Drying times may vary based on the environmental conditions.
9. Discard disposable wipes and gloves.
10. Start disinfection by putting on new disposable gloves.
11. Get a new wipe or dampen a soft, low-linting cloth with one of the permitted detergents.
12. Wipe the equipment surface. Make sure all surfaces are uniformly wiped. Treated surfaces must remain visibly wet for a minimum of two minutes (Super Sani-Cloth) or according to the disinfectant manufacturer's instructions. To ensure this, use additional disinfectant wipes as necessary.
13. Follow steps 5-9 to complete the disinfection.

Optional Kits List

To order parts, contact your local GE HealthCare representative. Contact information is available at <http://www.gehealthcare.com>. Make sure you have all necessary information at hand.

Part number	Description
6660123-73	75mm holder Kit (75mm height, 5pcs/box)

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